



**Animal Advanced
Treatment Solution**

Instructions for Use



Sil.Vet OTC is a buffered solution, with a pleasant odor, for the wound management, disinfection and moisturization. Sil.Vet OTC is a clear, ranging from colorless to pale yellowish solution, protected from microbial contamination by a low concentration of ionic silver and supplied in 100mL, 250mL or 500mL bottles.

INTENDED USE AND INDICATIONS FOR USE

Sil.Vet OTC is intended for use in animals and pets, for the management and moisturizing of minor cuts, minor burns, abrasions and irritated areas.

Sil.Vet OTC Solution is ideal for in-house treatment of wounds and is an integral part of any first aid kit for pets, horses, and other animals. It is effective in treating hot spots, abrasions, sores, cuts, rashes, burns, and skin irritations.

GENERAL DIRECTIONS FOR USE

1. Clean the affected area gently.
2. Apply Sil.Vet OTC directly on the wound or surrounding skin using a spray, syringe, foam bottle, or gauze. Ensure coverage beyond wound edges and allow to dry. No rinsing is needed.
3. For bandaging, apply Sil.Vet OTC to gauze and secure with a secondary dressing.

Apply 1-2 times daily until the wound heals or desired results are achieved. Between dressing changes, moisten dressings with Sil.Vet OTC for easier removal.

Use up to 10mL per kg of body weight in 24 hours. Sil.Vet OTC is a stand-alone solution suitable for removing crusty or moistening dressings.

DO NOT wash the wound with Saline, Before or After using Sil.Vet OTC as it may reduce the solution's efficacy.

DO NOT INJECT.

.



CAUTIONS AND WARNINGS

- For external use only. Avoid contact with eyes.
- Use only undamaged bottle.
- Do not use in patients with history of hypersensitivity to any of the ingredients.
- Discontinue use if irritation develops.
- Avoid usage of turbid or intensely colored solution.
- Protect bottle from direct contact with the wound.
- Bottle that comes in direct contact with the wound must be discarded after use.
- For single patient use only.
- Store between 10°C-30°C (50°F-86°F) in a dry place.
- If the bottle is closed immediately after use and the top of the bottle is protected from contamination, the solution may be reused within 28 days from first opening.
- The empty bottle can be disposed to the regular trash can. A bottle that contains solution should be cleared off to an appropriate site.

.INGREDIENTS

Ingredients include water for injection (WFI), glycerol (glycerin), sorbitan monolaurate (Tween-20), TRIS, menthol and silver nitrate 0.01% (w/v).

Each ml contains ionic silver in the form of silver nitrate 0.01% (w/v).

Each package contains (12) bottles of 500 mL or 250mL, or twenty four (24) bottles of 100mL and instructions for use leaflet, which is part of the bottle label.

SYMBOLS

The following provides a description of the graphic symbols that appear on the Sil.Vet label:

	Caution
 eIFU indicator	Consult instructions for use
	Keep bottles out of direct sunlight
	Temperature Limitation
	Single patient - multiple use
	Sterilized using aseptic processing techniques
	Do not resterilize. Resterilization may reduce the effectiveness of the product
	Do not use if package is damaged
	Keep out of reach of children
	19G needle (application pressure of ~ 10 psi)
	Medical device
	Batch code / LOT #
	Date of manufacture Country of manufacture
	Use by date
	Catalog number
	Manufacturer
	Distributor
	UDI - Unique device identifier

Global Wholehealth Partners Corp.
8025 Summer Place St.
Fontana, Ca. 92336
Phone:(833)773-4947
www.silvet.gwhpcorp.com