



# Wound Management Case Study#5

---

DVM. AMOS FRISHLING

Case Study: Treatment  
of Canine Ear Lesion  
Using Sil.Vet Solution



# Wound Management Case Study #5

**Patient:** Female dog (name: Miley)  
**Injury Location:** Outer edge of the right ear  
**Treatment Period:** 1 week

Case Study: Treatment  
of Canine Ear Lesion  
Using Sil.Vet Solution

2023

- Treatment Protocol:**
- Sil.Vet solution applied twice daily
  - No dressing applied
  - Wound cleaned gently with Sil.Vet at each application

| Initial Condition (Before Treatment)                                                                                                                                                                                                                                                                                                                                          | Pic.1                                                                                |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
| <p>A visible lesion on the edge of the ear with a mild area of hair loss and skin irritation</p> <p>No bleeding, but the area appeared dry and slightly rough in texture</p> <p>No signs of infection, but potential for scratching or further damage if untreated</p>                                                                                                        |   |
| Progress After 1 Week of Sil.Vet Use (Twice Daily)                                                                                                                                                                                                                                                                                                                            | Pic.2                                                                                |
| <p>The ear shows <b>significant improvement</b></p> <p>The lesion appears <b>less irritated</b>, with <b>reduced redness</b></p> <p>Hair regrowth is starting at the previously affected area</p> <p>The wound looks <b>clean, dry, and well-healed</b></p> <p>No scabs or signs of infection were observed</p> <p>The dog displayed <b>no discomfort</b> during handling</p> |  |

## Conclusion

The use of **Sil.Vet solution** twice daily, without dressings, proved to be an **effective and gentle treatment** for a superficial ear lesion in a dog.

### Benefits Observed:

- Simple, non-invasive application
- Promoted healing and skin regeneration
- Safe and well-tolerated by the animal
- No adverse reaction noted